





"Perfection consists not in doing extraordinary things, but in doing ordinary things extraordinarily well"

- Angelique Arnauld -





EdelweissDR is introducing a new level in direct esthetic restorations with its inspiro direct system, developed under the guidance of Dr. Didier DIETSCHI. This material expands the possibilities in free-hand bonding techniques following the well established "NATURAL LAYERING CONCEPT" leading to uncompromised esthetic and functional results thanks to the new, improved homogenous nano-hybrid technology combining surface smoothness and mechanical strength...

Try and convince yourself!



Natural Layering Concept

The concept was introduced in 1995 by Prof. Dr. D. Dietschi and published for the first time in 1997. It is based on the idea of approaching nature as closely as possible, using a simplified and more reliable layering technique with two layers only (dentin & enamel) to perfectly mimic the tooth structure and appearance. **inspiro** composite system represents the latest development in the application the Natural Layering Concept, with improved optical properties and a more comprehensive, logical choice of color and material consistencies to obtain restorations of unsurpassed esthetic integration and quality.

A COMPREHENSIVE ASSORTMENT OF DENTIN & ENAMEL SHADES FOR A 2-LAYER APPROACH

intensity levels in **BODY SHADES** for covering all variations in dentin chroma. The system offers 6 chroma levels for the restorative material and 7 chroma levels for the flowable material. Intensity 0 was developed for bleached teeth and masses with chroma intensity extending beyond the VITA color range (intensity 5 to 7) for optimal integration of class V restorations or for dentin mamelons. This comprehensive choice of body shades will allow the clinician to cover all dentin chroma variations found in nature. All body shades exhibit the same color (hue) and a similar opacity and fluorescence, greatly simplifying shade selection, as compared to other systems.

tints and translucency levels in **SKIN SHADES** for mimicking all natural enamel variations, which extend esthetic possibilities and system reliability. BLEACH and WHITE skin shades provide the brightness and lightness needed for young patients and rejuvenated smiles; skin NEUTRAL is a very common enamel tint to be used for adult patients, while skin IVORY and TRANSPARENT complete the enamel shades for mature teeth which require more depth and warm tints. 3 out of the 5 restorative skin shades are also available in a flowable consistency.





...layering does not need to be complicated to be highly esthetic!



DIRECT RESTORATIVES

inspiro comprehensive color range & shade guide

The **inspiro** direct shade guide allows a precise and reliable color evaluation. It has been developed as an integrant tool of the "Natural Layering Concept" and includes 2 distinctive restorative layers (body & skin). It has smaller dentin/body samples, which can be inserted into enamel/skin samples to demonstrate any 2-LAYER combination. The selection of enamel/skin is to be made visually where more enamel can be observed, namely in the incisal or proximal areas. It is recommended to use glycerin in between both composite samples.



VITA	A1	B1	C1	A2	B2	C2				A3.5	A4	В4	C4
inspiro Body	i1	i1	i1	i2	i2	i2	i3	i3	i3	i4	i5	i5	i5
inspiro	W/N	1	Т	W/N	1	Т	N	1	Т	N	N	1	Т





Step 1: selection of dentin chroma in the cervical area

Step 2: combination of dentin & enamel samples











Teeth can present unlimited color and light variations. For the large majority of them, the basic application of the "Natural Layering Concept" will perfectly render the beauty of natural tissues. For the few cases which require additional coloring, the **inspiro** system provides all needed effect shades, which can be used for the following characterization:

serves to enhance blue-opalescent effects of the incisal edge in front teeth. It is not too intense so that over-characterization can be avoided

is used to increase chroma locally when required. It is applied over the normal dentin build-up to mimic all kind of "orange" intensive effects such as dentin sclerosis, counter-opalescence and some forms of pigmentation, due for instance to aging

helps to simulate widespread enamel opacities and serves then to create those discrete, natural "imperfections" such as light fluorosis or surface demineralization











is mainly used in combination with other effect shades (i.e.: ICE or CHROMA) to produce "opaquers" of desired shade and chroma to cover either severely discolored tissues or metals

is to be placed in the depth of occlusal fissures of class I and II restorations to improve their esthetic integration and stimulate the artistic talent of practitioners when doing posterior restorations









NANO PARTICLES


A new homogenous Nano-Hybride Composite:

Fillers with particle sizes in the nanometer range ("nano particles") show a strong tendency to aggregate and agglomerate. In this "cluster" form, they do not reach their full potential for improvement of the mechanical and surface properties of the composite; in fact, such aggregated and agglomerated nano particles have been known for a long time and used in dental materials for many years (impression materials and composites for instance). inspiro, on the contrary, uses a new technology which takes advantage from true nano-particle technology where, differently from all other so called "inhomogeneous" nano-hybrid composites, this ultrafine filler is separated and evenly distributed in the matrix. Here, the nano-particles can reach their full potential for enhancement of surface quality, flexural strength and modulus, among many other unique physicochemical properties.

LINEAR SHRINKAGE DISPLACEMENT

(University of Geneva; according to the methodology of Stavridakis et al, 2003 & 2005)

um @ 3 minutes for 1mm composite thickness

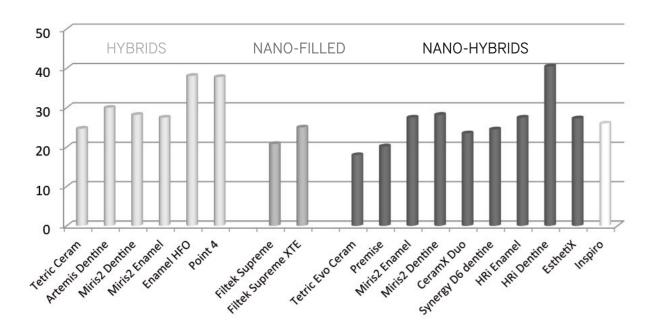

Mechanical properties & benefits:


- · Low shrinkage due to nano-technology and high amount of filler, 83%
- Very good physical and mechanical properties
- Good abrasion resistance
- Antibacterial surface due to zinc and fluoride particles in the filler
- Excellent polishability and gloss retention
- Natural fluorescence and opalescence


(Source: internal data from EdelweissDR)


FLEXURAL MODULUS (GPa)

FLEXURAL STRENGTH (MPa)

COMPRESSIVE STRENGTH (MPa)

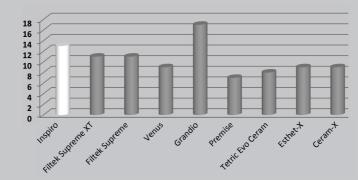

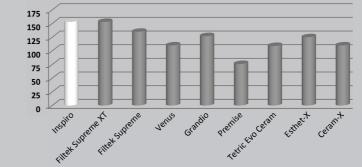

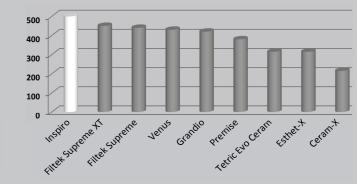

Properties	Restorative shades	Flowable shades
Filler content	83% (W) / 65% (V)	65.5% (W) / 38% (V)
Particle size range	0.02-3 μm	0.02-3 μm
Glass filler type	Barium glass with Zinc-Fluoride & nanoparticles	Barium glass with Zinc-Fluoride & nanoparticles
Glass average size	0.7µm	0.7µm
Matrix system	Bis-GMA	Bis-GMA
Flexural modulus	12.5 GPa	6 GPa
Flexural strength	150 MPa	120 MPa
Compressive strength	480 MPa	350 MPa
Working time	~1 min	~1 min



Form Corrections



DIRECT RESTORATIVES















Pirect Veneers

..."simple and effective layering approach" does not mean to accept any compromise on the esthetic treatment outcome, on the contrary!





"What do I find so special about the inspiro system?"

1. The **inspiro** system is based on a simple and rational shading concept, consisting of universal dentin shades and multi-tint, multi-translucency enamels, which I have refined over the last decade. Using this straightforward approach, I am able to achieve the right combination for mimicking any natural tooth appearance.

- 2. The **inspiro** effect shades give even more freedom to develop our "artistry" and achieve unsurpassed esthetic results following a simple, effective layering approach
- 3. I can pre-visualize and control the color integration thanks to the special bi-laminar **inspiro** shade guide; it means that when the **inspiro** shade sample matches the natural teeth, I will get it right!
- 4. I have an accurate selection of restorative and flowable consistencies suitable for various different clinical situations, from posterior restorations to more complex smile rehabilitations... effectively

For once, clinical excellence can be the result of simplicity and reliability!

Excellence and Simplicity in es

Dr. Didier Dietschi





















Esthetic Rehabilitation



MASTER KIT REF 3010	INTRO KIT REF 3030	INSPIRO COMPULE TOOLBOX	MASTER KIT REF 3020	INTRO KIT REF 3040	INSPIRO SYRINGE TOOLBOX
90	70	Composite Dentin Compules	6	4	Composite Dentin Syringes
10	-	inspiro Body i0 Compule 0.3 g	1	-	inspiro Body i0 Syringe 3 g
15	15	inspiro Body i1 Compule 0.3 g	1	1	inspiro Body i1 Syringe 3 g
20	20	inspiro Body i2 Compule 0.3 g	1	1	inspiro Body i2 Syringe 3 g
20	20	inspiro Body i3 Compule 0.3 g	1	1	inspiro Body i3 Syringe 3 g
15	15	inspiro Body i4 Compule 0.3 g	1	1	inspiro Body i4 Syringe 3 g
10	-	inspiro Body i5 Compule 0.3 g	1	-	inspiro Body i5 Syringe 3 g
80	60	Composite Enamel Compules	5	3	Composite Enamel Syringes
20	20	inspiro Skin Bleach Compule 0.3 g	1	1	inspiro Skin Bleach Syringe 3 g
20	20	inspiro Skin White Compule 0.3 g	1	1	inspiro Skin White Syringe 3 g
20	20	inspiro Skin Neutral Compule 0.3 g	1	1	inspiro Skin Neutral Syringe 3 g
10	-	inspiro Skin Transparent Compule 0.3 g	1	-	inspiro Skin Transparent Syringe 3 g
10	-	inspiro Skin Ivory Compule 0.3 g	1	-	inspiro Skin Ivory Syringe 3 g
7	3	Composite Dentin Flowable	7	3	Composite Dentin Flowable
1	1	inspiro Flowable Body i1 Syringe 1.5 g	1	1	inspiro Flowable Body i1 Syringe 1.5 g
1	1	inspiro Flowable Body i2 Syringe 1.5 g	1	1	inspiro Flowable Body i2 Syringe 1.5 g
1	1	inspiro Flowable Body i3 Syringe 1.5 g	1	1	inspiro Flowable Body i3 Syringe 1.5 g
1	-	inspiro Flowable Body i4 Syringe 1.5 g	1	-	inspiro Flowable Body i4 Syringe 1.5 g
1	-	inspiro Flowable Body i5 Syringe 1.5 g	1	-	inspiro Flowable Body i5 Syringe 1.5 g
1	-	inspiro Flowable Body i6 Syringe 1.5 g	1	-	inspiro Flowable Body i6 Syringe 1.5 g
1	-	inspiro Flowable Body i7 Syringe 1.5 g	1	-	inspiro Flowable Body i7 Syringe 1.5 g
3	2	Composite Enamel Flowable	3	2	Composite Enamel Flowable
1	-	inspiro Flowable Skin Bleach Syringe 1.5 g	1	-	inspiro Flowable Skin Bleach Syringe 1.5 g
1	1	inspiro Flowable Skin White Syringe 1.5 g	1	1	inspiro Flowable Skin White Syringe 1.5 g
1	1	inspiro Flowable Skin Neutral Syringe 1.5 g	1	1	inspiro Flowable Skin Neutral Syringe 1.5 g
5	3	Effect Shades	5	3	Effect Shades
1	1	inspiro Effect Shade Azur Syringe 1.5 g	1	1	inspiro Effect Shade Azur Syringe 1.5 g
1	-	inspiro Effect Shade Ice Syringe 1.5 g	1	-	inspiro Effect Shade Ice Syringe 1.5 g
1	-	inspiro Effect Shade Opaque Syringe 1.5 g	1	-	inspiro Effect Shade Opaque Syringe 1.5 g
1	1	inspiro Effect Shade Chroma Syringe 1.5 g	1	1	inspiro Effect Shade Chroma Syringe 1.5 g
1	1	inspiro Effect Shade Fissure Syringe 1.5 g	1	1	inspiro Effect Shade Fissure Syringe 1.5 g
		Accessories			Accessories
1	1	inspiro Shade Guide	1	1	inspiro Shade Guide
20	20	inspiro Application Tips	20	20	inspiro Application Tips

TOOLBOXES

23

REFILLS





COMPOSITE DENTIN COMPULES REFILLS COMPOSITE DENTIN FLOWABLE REFILLS inspiro Body iO Refill Compule 10 x 0.3 g 3610 inspiro Flowable Body i1 Refill Syringe 1 x 1.5 g inspiro Body i1 Refill Compule 10 x 0.3 g inspiro Flowable Body i2 Refill Syringe 1 x 1.5 g inspiro Body i1 Refill Compule 20 x 0.3 g inspiro Flowable Body i3 Refill Syringe 1 x 1.5 g 10 x 0.3 g inspiro Body i2 Refill Compule inspiro Flowable Body i4 Refill Syringe 1 x 1.5 g inspiro Body i2 Refill Compule inspiro Flowable Body i5 Refill Syringe 20 x 0.3 g 1 x 1.5 g inspiro Body i3 Refill Compule 10 x 0.3 g inspiro Flowable Body i6 Refill Syringe 1 x 1.5 g inspiro Body i3 Refill Compule 3670 inspiro Flowable Body i7 Refill Syringe 20 x 0.3 g 1 x 1.5 g inspiro Body i4 Refill Compule 10 x 0.3 g inspiro Body i4 Refill Compule 20 x 0.3 g inspiro Body i5 Refill Compule 10 x 0.3 g COMPOSITE ENAMEL FLOWABLE REFILLS 3700 inspiro Flowable Skin Bleach Refill Syringe 1 x 1.5 g inspiro Flowable Skin White Refill Syringe COMPOSITE ENAMEL COMPULES REFILLS 1 x 1.5 g 3720 inspiro Flowable Skin Neutral Refill Syringe 1 x 1.5 g inspiro Skin Bleach Refill Compule 10 x 0.3 g inspiro Skin Bleach Refill Compule 20 x 0.3 g inspiro Skin White Refill Compule 10 x 0.3 g EFFECT SHADE REFILLS inspiro Skin White Refill Compule 20 x 0.3 g inspiro Skin Neutral Refill Compule 10 x 0.3 g 3800 inspiro Effect Shade Azur Refill Syringe 1 x 1.5 g inspiro Skin Neutral Refill Compule inspiro Effect Shade Ice Refill Syringe 20 x 0.3 g 1 x 1.5 g inspiro Skin Transparent Refill Compule 10 x 0.3 g inspiro Effect Shade Opaque Refill Syringe 1 x 1.5 g inspiro Skin Ivory Refill Compule inspiro Effect Shade Chroma Refill Syringe 10 x 0.3 g 1 x 1.5 g inspiro Effect Shade Fissure Refill Syringe 1 x 1.5 g COMPOSITE DENTIN SYRINGES REFILLS ACCESSORIES inspiro Body iO Refill Syringe 1 x 3.0 g 3900 inspiro Shade Guide inspiro Body i1 Refill Syringe 1 x 3.0 g inspiro Body i2 Refill Syringe 1 x 3.0 g 3920 inspiro Application Tips Refill 20 pieces inspiro Body i3 Refill Syringe 1 x 3.0 g inspiro Body i4 Refill Syringe 1 x 3.0 g inspiro Body i5 Refill Syringe 1 x 3.0 g COMPOSITE ENAMEL SYRINGES REFILLS 3500 inspiro Skin Bleach Refill Syringe 1 x 3.0 g inspiro Skin White Refill Syringe 1 x 3.0 g

1 x 3.0 g

1 x 3.0 g

1 x 3.0 g

DIRECT RESTORATIVES

inspiro Skin Neutral Refill Syringe

inspiro Skin Ivory Refill Syringe

inspiro Skin Transparent Refill Syringe



Edelweiss DR AG
Unter Altstadt 28, Mercandor
6300 Zug
Switzerland
Tel. +41 41 933 33 77
+41 41 911 33 77
Fax +41 41 370 33 77
info@edelweissdr.com
www.edelweissdr.com



Art pictures courtesy of Dudu Meideros www.clickdudu.com

Clinical Cases courtesy of Dr. Didier Dietschi www.edudentinternational.com

